

BD110真空測試紙

BOWIE-DICK TEST SHEETS

使用說明

使用指示: INDICATIONS FOR USE

BD110真空測試紙是設計用來在134°C操作測試供每天測試監控預抽真空的蒸汽滅菌器的。BD110是根據在AAMI ST79中所描述設計供在醫療院所所打包的bowie-dick波伊-迪克試驗布包中使用的。

顏色改變: COLOR CHANGE

BD110真空測試紙當一起暴露在蒸汽消毒滅菌週期與可接受的真空週期時BD110真空測試紙會從藍色變成黑色。

關鍵的參數: CRITICAL PARAMETERS

在134°C蒸汽消毒滅菌一起作用3.5-4分鐘。

使用說明: INSTRUCTIONS FOR USE

1. 包裝一個bowie-dick波伊-迪克毛巾測試布包並且放一張BD110真空測試紙到中心。
2. 執行一個縮短的蒸汽消毒滅菌週期(省略乾燥的階段)適當地加熱消毒滅菌鍋。
3. 將毛巾測試布包放在滅菌鍋的底部接進排水口的地方。
4. 根據製造商的指示執行bowie-dick波伊-迪克真空測試週期。
5. 在134°C中執行測試3.5分鐘。這個測試時間絕不可超過4分鐘。
6. 2到5分鐘短的乾燥的時間不會影響試驗。

判讀: INTERPRETATION

1. 從滅菌鍋中取出毛巾布包並且立即打開。
2. 從毛巾布包中取出BD110真空測試紙。
3. 如果滅菌鍋功能正常，BD110真空測試紙指示圖表將顯示整個一致相同的顏色改變(黑色)。
4. 當明顯有殘餘的空氣存在時，這個指示圖表將在中心顯示一個藍色的區域。

安全預防措施: SAFETY PRECAUTIONS

小心：毛巾布包裝是很熱的必需小心打開避免燙傷。

儲存: STORAGE

- 請將BD110真空測試紙儲存在0°- 40°C正常的室溫以及10-60%相對濕度中。
- 避免水分和過高的濕度。
- 避光保存測試過的試紙防止轉回原來的藍色。

INTERFERING SUBSTANCES OR CONDITIONS

There are NO KNOWN INTERFERING SUBSTANCES OR CONDITIONS that could affect the intended use of the indicator or adversely affect the indicator performance.

RELEASE OF TOXIC SUBSTANCES


The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

DECLARATION OF CONFORMITY

Steri-Pak LF Bowie-Dick Test Sheet indicator conforms to all applicable portions of ISO 11140-1 pertaining to Class 2 indicators for use in specific tests.

Steri-Pak LF Bowie-Dick Test Sheets have been tested according to ISO 11140-3 and fully complies with the performance requirements after exposure to dry saturated steam when tested under the conditions specified in ISO 11140-3.

藍色變成黑色

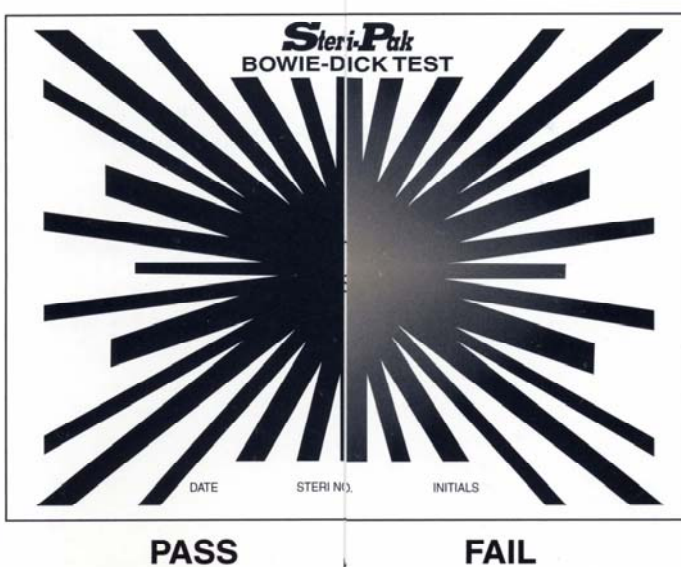

穗滿企業股份有限公司

台北市忠孝東路4段162號5樓之3

服務電話：(02) 7317727

傳 真：(02) 7318221

www.bge.com.tw

bright@bge.com.tw

中華民國 九十八年七月七日校譯